

Patient Lifting Equipment

Activity		Type Of Service				
		A	B	C	D	E
Last Issue Date: 04/03/2019						
1	Check the hanger frame for any damage, wear or irregularities.				Y	
2	Check the spreader bar for any damage, wear or irregularities.				Y	
3	Check the jib including the pivot points for any damage, wear or irregularities.				Y	
4	Check the mast / pillar for any damage, bending or irregularities.				Y	
5	Check the condition of the lift bands.				Y	
6	Check the base / chassis for any damage or irregularities.				Y	
7	Check the leg pivots for any damage, wear or irregularities.				Y	
8	Check the front and rear castors for any damage, wear or irregularities.				Y	
9	Check the electrics / looms for any damage, wear or irregularities.				Y	
10	Check the condition and operation of the motor / actuator.				Y	
11	Check the condition and operation of the hydraulics.				Y	
12	Check the batteries & charger.				Y	
13	Check the condition and operation of the handset.				Y	
14	Check the condition and operation of the scales.				Y	
15	HOIST SLINGS AND STRAPS - Refer to Special Comments for the C service notes (EQ249C) and patient pre-use requirements. Check the following: a) The lifting slings and straps for damage, wear or any damaged stitching; b) The connections between the lifting straps and the hanger; c) Sling is the correct type to be used / compatible with the hoist equipment. (Note: Increase inspection frequencies where the unit is used or cleaned more frequently than normal).			Y	Y	
16	Check the condition and operation of the accessories.				Y	
17	HOIST WEIGHT TEST: Undertake a hoist weight test with the rated load ensuring the lift is effortless and without any jarring sounds.				Y	
18	If applicable, hoists used in wet areas are to have an annual reapplication of a moisture protection coating.				Y	
19	Apply an inspection sticker (label) with the date of the test and next test due date to the hoist (include tested weight), body support unit and trolley (if applicable).				Y	
20	Record the results in the customer service report / log book. Include: inspection date; next inspection due date; identification details / serial numbers; condition; works undertaken; and identification and signature of the inspector.			Y	Y	

**Disclaimer** - This TDS is intended to provide guidance only to support Preventative Maintenance servicing activities. All TDS users are encouraged to read and understand the full [Conditions of Use](#) provided on the final page of this document and on the DPTI website.

For more information please contact the applicable Facilities Management Service Provider.



Government of South Australia  
Department of Planning,  
Transport and Infrastructure

Download Date: 22/12/2020

## Special Comments and Technical Data

---

C SERVICE 6 MONTHLY (MINIMUM) D SERVICE ANNUAL  
THE PREVENTATIVE MAINTENANCE SERVICES ARE TO COMPLY WITH AS10535 (REFER ANNEX B FOR INFORMATION ON THE INSPECTIONS) & THE MANUFACTURER'S DOCUMENTATION & SUIT SITE CONDITIONS.  
EQ249 IS A PRIMARY TDS. FOR THE SPLIT SERVICE TDSS REFER TO: EQ249C (C SERVICE) & EQ249D (D SERVICE).  
EQ249 IS TO BE SCHEDULED IN CONJUNCTION WITH EQ375: SERVICE OF THE RAIL SYSTEM.  
IMMEDIATELY NOTIFY THE SITE REPRESENTATIVE AND THE FACILITY MANAGER OF ANY WEAR, DAMAGE OR DEFECTS THAT MAY HAVE SAFETY IMPLICATIONS & REMOVE THE HOIST FROM SERVICE UNTIL RECTIFIED  
NOTE: FOR PATIENT PRE-USE REQUIREMENTS:  
PRIOR TO EVERY USE OF THE HOIST THE SITE STAFF ARE TO UNDERTAKE THE VISUAL INSPECTION CHECKS LISTED IN ACTIVITY 15.

**Disclaimer** -This TDS is intended to provide guidance only to support Preventative Maintenance servicing activities. All TDS users are encouraged to read and understand the full [Conditions of Use](#) provided on the final page of this document and on the DPTI website.

*For more information please contact the applicable Facilities Management Service Provider.*



**Government of South Australia**  
Department of Planning,  
Transport and Infrastructure

**Download Date:** 22/12/2020

2 of 3

# Conditions of Use

A TDS should be used as a generic guide for Facilities Management Service Providers (FMSP) to deliver Facilities Maintenance Services, specifically Preventative Maintenance in accordance with the DPTI [Agency Work Procedure Manual](#).

Users of a TDS should familiarise themselves with the following **Conditions of Use**:

1. You use the information and data contained in any TDS published by DPTI at your sole risk. DPTI-AGFMA does not have the expertise to provide complete or accurate technical data schedules and provides these technical data sheets merely as a starting point and/or guide.
2. If you rely on the information in a TDS you are responsible for ensuring, by independent verification, its accuracy, currency and completeness. DPTI cannot guarantee that the information contained in a TDS meets the standards or requirements of legislative requirements.
3. A TDS is not an exhaustive list of tasks or obligations that may be required and is generally generic in nature. DPTI does not oblige the user of a TDS to rely on it to the exclusion of other sources of information. For example, manufacturer's requirements may need to be followed for additional and / or alternative tasks and for additional and / or alternative servicing frequencies. You should always check the primary source of information such as the Australian Standards, manufacturer's specifications, legislation and other standards before undertaking any work to which a schedule may apply.
4. You must apply appropriate risk management principles and carry out all tasks in accordance with obligations under the [Work Health and Safety Act 2012](#).
5. You must not use a TDS unless you meet the specific competency requirements for preventative maintenance servicing works and are capable of applying the TDS to your trade or area of expertise. A TDS is only to be used by people who are specifically instructed by DPTI to use them and only for work undertaken in South Australia.
6. DPTI, its agents, instrumentalities, officers and employees make no representations, express or implied, as to the accuracy of the information, the data contained in a TDS or the suitability of a TDS for a particular purpose.
7. DPTI does not provide legal advice. DPTI accepts no liability, howsoever arising, for any loss resulting from the use of a TDS and any information or data or reliance placed on them.
8. DPTI may change information and data in a TDS without notice.
9. DPTI may revise this disclaimer at any time by updating these Conditions of Use.